

Topi-Click™ Dispensing Instructions


Read all of this information and instructions carefully before dispensing cream from the Topi-Click™.

1. Pop off the cap by placing thumb in groove on the front and pressing upward.
2. Turn the base in direction shown below in picture #2 to dispense one dose or number of clicks per topical providers instructions.
3. Apply cream by rubbing applicator pad to area of skin as directed by topical provider.
4. Replace cap.
5. Once you've completed all of your prescribed doses, dispose of the Topi-Click™—it is not refillable.

***The Refill Indicator:** If your prescription has refills, the Topi-Click™ will indicate when it's almost empty and time for you to call the pharmacy to get your refill ready for pick-up. The refill indicator is on the Topi-Click™ cylinder near the top. When the blue or pink colored indicator inside the cylinder rises to the word "REFILL," it's time to call the topical provider for a refill. **See The Refill Indicator Label below.*



*The Refill Indicator Label

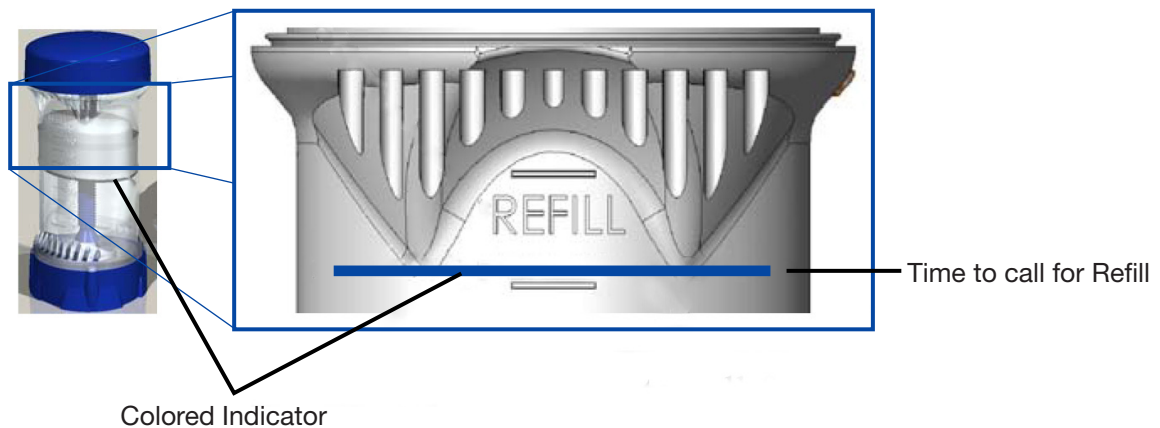

The Topi-Click™ product may be improved over time based on customer feedback on usage. Custom Rx TDA, LLC and/or the patent holders are not liable for any damages arising from the products dispensed in the Topi-Click™ or its improper use.